



living skintm


touch
bionics

my livingskin Quick Reference Guide

Kit Contents

- iPad® Air with my livingskin color capture system
- Cleaning cloth for iPad Air
- iPad Air charger
- OttLite
- OttLite spare bulb
- Grey plate
- White plate
- Return packaging and label (North America Only)

Getting Started

To use my livingskin, please start by following the instructions below:

1. Have your Touch Bionics order number available. Order number can be found on order acknowledgement, packing slip or is available from Customer Support
2. Ensure you have ample working space and controlled lighting
3. Clean the iPad Air screen and camera lens with provided cleaning cloth
4. Power on the iPad Air and start the application by tapping the my livingskin icon shown in Figure 1*
5. Follow the onscreen instructions to process your order (Figure 2)
6. Complete impressions with provided impression kit

Note: The distal end of the patients limb should always be pointing towards the OttLite when taking photos

Note: Ensure impressions are taken after color capture process as impression material can change skin tone

*You will need a stable data or Wi-Fi connection

Figure 1


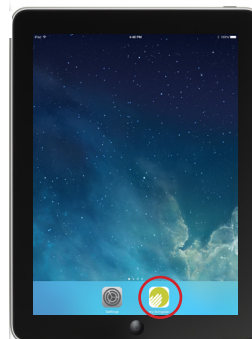



Figure 2

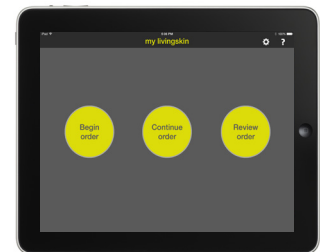

Lighting & Image Capture

Upon opening the application, you will be prompted to review the proper setup instructions in advance of capturing images. An example is also included below:

Figure 3

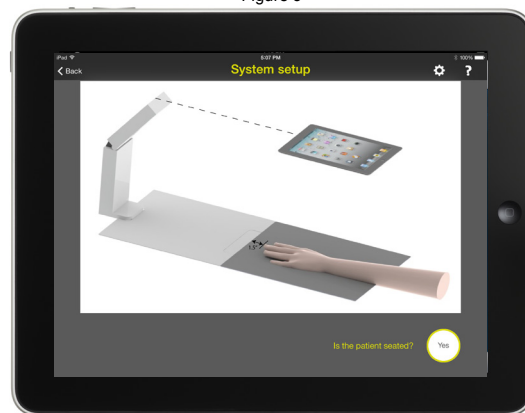

Support

Please refer to www.touchbionics.com/downloads/document-library to ensure you have the latest supporting documentation, including the Field Impression Guide and livingskin User Manual.

If you have any questions, please contact Customer Support.

North American Customers
(Canada, Mexico & US)
Tel: +1-855-694-5462

UK & Non-North American Customers
Tel: +44 1506 438 556



MA 01255, Issue No. 2, May 2014