



virtu-limb<sup>tm</sup>

  
touch  
bionics

# virtu-limb™ Quick Reference Guide

---

## Downloading biosim & system requirements

To register for biosim, please go to [www.touchbionics.com/biosimdownload](http://www.touchbionics.com/biosimdownload)

- Windows XP, Windows Vista, Windows 7, Windows 8, or Mac OS X Lion (10.7 and higher)
  - Microsoft .NET Framework v3.5
  - USB port for Bluetooth® connector
  - Administrative rights to install the software and connector
- 

## Downloading the biosim mobile app

biosim mobile control app is available on the Apple® App store for download and is compatible with iPhone® (4s or newer), iPad® (4th generation or newer), iPad® mini, iPod (5 or newer).

---

## Product Manuals

Please refer to [www.touchbionics.com/downloads/document-library](http://www.touchbionics.com/downloads/document-library) to ensure you have the latest copy of the product manual. To request a printed manual, please contact Customer Support.

---

## Battery Charging

To ensure compatibility and optimum performance use ONLY the Touch Bionics charger as supplied with your device.



To ensure compatibility and optimum performance, ONLY use the Touch Bionics charger as supplied.



Do not operate virtu-limb with charger connected

---

## Safety Guidelines

Please read these guidelines and information before using virtu-limb. Failure to do so may lead to risk of product malfunction or even a potential hazard to your health, and may lead to an invalidation of the warranty.

- Do not use this product near water.
- Do not expose the device to excessive amounts of moisture, liquid, dust, vibration, or shock.
- Do not expose the device to high temperatures.
- Do not expose the device to any form of open flame.
- Do not use this product in an area where a potentially explosive atmosphere exists.
- Do not attempt to disassemble or modify the product. Only qualified members of Touch Bionics or its partners may perform maintenance, repairs, or upgrades to the product. Failure to comply will invalidate the warranty.
- Always use the Touch Bionics equipment provided (electrodes, software, etc).

See [www.touchbionics.com/downloads/document-library](http://www.touchbionics.com/downloads/document-library) for further General Safety, EMC and Electrical information for i-limb products.

North American Customers  
(Canada, Mexico & US)  
Tel: +1-855-694-5462

UK & Non-North American Customers  
Tel: +44 1506 438 556

